

# Houdini Harness Model 27

## Installation and User Manual



- SIZE "A" - SMALL  
#4604-2702-017
- SIZE "B" - LARGE  
#4604-2703-017
- ANTI-ESCAPE BUCKLE  
A#4604-2702-026 B#4604-2703-026
- PEN/KEY RELEASE UPGRADE  
#4604-2805-031
- CUSTOM: \_\_\_\_\_

### IMPORTANT CONSUMER INFORMATION NOTICE:

This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

### SUPPLIER:

This manual must be given to the user of this product.

### USER:

Before using this product, read entire manual and save for future reference.

Instruction Video



medifab®

User Manual for all Houdini 27 Harnesses  
Manual: Houdini27201504

Serial Number: \_\_\_\_\_

# MODEL 27

Designed to offer postural support and a certain degree of restraint to children and adults on single or split backed double seats, to keep them in an up right sitting position.

Fits most types of vehicle seating including low or high backed

**The Model 27 is used on aircraft seats by many leading airlines.**

The two transverse fixing straps do not interfere with the passenger behind using the drop down trays.

We always advise our clients to seek permission from the airlines to use our harnesses prior to departure.

The harness consists of a full body harness connected to a back pad which is fitted horizontally around the vehicle seat back with two horizontal fixing straps.

**It is not a safety harness & should be worn in conjunction with the vehicle safety belt.**



27A - 5yrs to 8yrs



Model 27 Fixing Straps

The waist belt is fitted with the **Press Button** buckle as standard.

The **Steel Safety** buckle can be fitted for those passengers with challenging behaviour.

**Available in three sizes & a choice of two buckles:**

	<b>Standard Buckle</b>	<b>Steel Buckle</b>
Up to 5 yrs	27I	27ISB
5 yrs to 8 yrs	27A	27ASB
9 yrs to Adult	27B	27BSB



## Optional Extra's

A **Cross Strap** can be fitted to hold the shoulder straps together & help prevent them from slipping off the shoulders.

Two types available



Cross Strap



Cross Strap in Padded Sleeve


## **Crelling Harnesses For Disabled Ltd.**

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: [info@crelling.com](mailto:info@crelling.com) Website: [www.crelling.com](http://www.crelling.com)

# MODEL 27 HARNESS

BEING FITTED TO A MONARCH AIRCRAFT SEAT

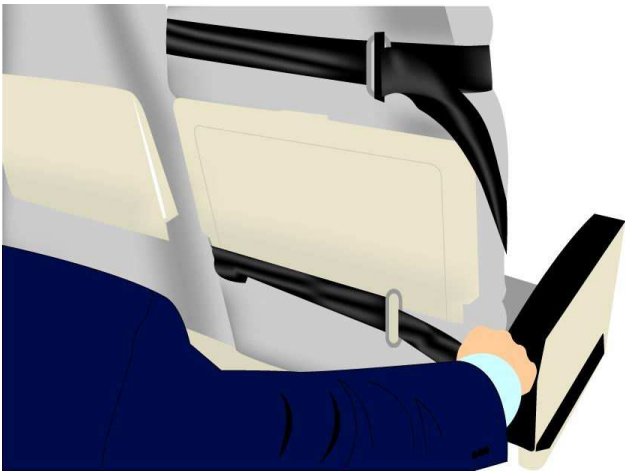

Accepted for passengers with special needs by many airlines  
Including:

